INSTRUCTIONS FOR USE

JOBST® UlcerCARE™

Made-to-measure

EN Instructions For Use

DE Anwendungshinweise

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SV Bruksanvisning

NO Bruksanvisning

DA Brugsanvisning

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SL Navodila za uporabo

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EN

Dear Customer,

Thank you for purchasing JOBST®, the medical gradient compression hosiery collection that is recommended by physicians.

JOBST® UlcerCARE™ is designed for maximum patient compliance for the effective management of venous leg ulcers. JOBST® UlcerCARE™ is an efficacious and easy-to-use compression system for the management of venous leg ulcers. JOBST® UlcerCARE™ also helps prevent recurrence of venous leg ulcers. Two components, a Medical Stocking (with or without a zipper) and a Compression Liner, provide compression with decreasing pressure gradient from distal to proximal.

1. Indications

The JOBST® UlcerCARE™ compression system is indicated for the following conditions:

- Management of venous leg ulcers after edema reduction
- Helps prevent recurrence of venous leg ulcers

2. Contraindications

The JOBST® UlcerCARE™ compression system is contraindicated when any of the following conditions are present:

- Ischemia (e.g. advanced arterial disease)
- Untreated septic phlebitis
- Uncontrolled congestive heart failure
- Phlegmasia cerulea dolens
- Incompatibility to fabric

3. Cautions

- Skin infections
- Severely compromised skin sensibility and impaired sensitivity of the limb
- Weeping dermatoses
- Immobility (confinement to bed)
- Conditions in which increased venous and lymphatic return is not desired.
- Open wounds must be covered with an appropriate dressing before the compression liner is put on.
- Appropriate foot wear must be worn when walking to help prevent falls.
- JOBST® UlcerCARE™ is recommended following edema reduction by the use of short stretch compression bandaging.
- While the Compression Liner may be worn 24 hours a day, the patient must remove the Medical Compression Stocking during bed rest.
- The Compression Liner should be changed in accordance with daily hygiene and wound inspection needs.
- Appropriate hygiene and skin care should be applied.
- After complete healing of the venous leg ulcer, continued use of JOBST® UlcerCARE™ or other appropriate Medical Compression Stocking (e.g. JOBST® Medical LegWear) is recommended to help prevent recurrence.
- Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also

considering age and health status.

- When re-ordering a garment we recommend remeasuring the patient as the limb circumference might have changed.
- Incorrectly measured and badly fitting products may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage during use.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.
- The attending physician decides whether compression in general can be applied to patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome, CRPS (M. Sudeck), malignant lymphedema, gangrene.
- Patient measurements should be taken by a trained person to determine the correct product size needed and to decide whether a made-to-measure or ready-to-wear garment is required.

Special notes:

- Caution: Long fingernails and jewelry may damage the stockings when putting them on. For easier application use rubber gloves or a donning aid.
- Do not bring the compression stockings into contact with greasy ointments, body lotions or similar substances unless recommended by JOBST®.
- Loose threads or loops should not be cut off, since this may cause holes or runs to form in the stockings.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state.
- JOBST® UlcerCare™ is intended for single patient use only.

INSTRUCTIONS FOR USE

EN

4. Application

The best way to put on your JOBST® stockings:

JOBST® UlcerCARE™ Compression Liner

- 
- Gather the Compression Liner in both hands, so that the toes can easily reach the end of the Liner.
- 
- Pull the Compression Liner onto the foot, aligning the heel of the Liner with the heel of the foot.
- 
- Gently pull the Compression Liner up the leg, taking special care to ease it over wound dressing.
- 
- Pull it upwards to the knee. Do not roll down or fold the top of the stocking.

- 
- Smooth any wrinkles and adjust the heel and toe of the Compression Liner for comfort.

JOBST® UlcerCARE™ Medical Compression Stocking without zipper

- 
- Apply the Compression Liner as instructed above. Start with the Medical Compression Stocking inside out. Tuck the foot part inside up to the heel and hold it with both hands, seam facing down.
- 
- Stretch your garment as wide as possible and in one motion, pull it onto your foot. Put the heel pocket over the back of your heel.
- 
- Take hold of the fabric just past your toes always at the sides, never the seams. Pull it up over your foot past the ankle. It should now be right side out. Adjust the garment so that the heel is in place.
- 
- Adjust the toe area for comfort by pulling your garment toward your toes and releasing.

- 
- Now work it all the way to the knee about 2" (5 cm) at a time, smoothing the creases as you go - adjust any wrinkles or uncomfortable spots by gently pulling the fabric until the pressure is even.

JOBST® UlcerCARE™ Medical Compression Stocking with zipper*

- 
- Apply the Compression Liner as instructed above. Open the zipper of the Medical Compression Stocking completely.
- 
- Point your toes and slip your foot into the foot area. Place your thumb between the zipper and your leg to protect your skin.
- 
- Zip it closed, keeping the elastic backing between your skin and the zipper.
- 
- Adjust any wrinkles or uncomfortable spots by gently pulling the fabric until the pressure is even.

*When selecting Left or Right Closure (zipper), choose the side opposite the wound.

5. Care instructions

JOBST® UlcerCARE™ compression system should, if at all possible, be washed daily, or at least every other day. For this purpose, you may use a regular mild detergent. Do not use laundry additives such as fabric softener, optical brightener or stain remover. Rinse your compression stockings well. Roll up the wet compression stockings in a towel after washing and press out excess moisture. Do not wring. Do not dry the compression stockings on a radiator or in direct sunlight. For tumble dryer compatibility please check your compression stocking's product label.

6. Directions for storage/Guarantee

JOBST® UlcerCARE™ compression system is intended for immediate use and no expiry date for storage is indicated. Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of six months. This assumes correct handling (e.g. product care, donning and doffing).

Please protect your JOBST® UlcerCARE™ compression system from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature. If the vascular disease persists after 6 months, please consult your attending physician concerning the possible need for a new compression stocking.

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Thank you for your trust.
Discover more about healthy legs and compression products on:
www.jobst.com

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34802-99433-00 JBR13759.01 © 2019 BSN medical Inc. Revision: 08/2019

	Artwork ID: JBR13759.01	Review Loop: 3	Printing Colours:  Pantone 281 C
Project Title: C2365_713 – JOBST® UlcerCARE™ Custom-Made IFU EMM		Date of Review Loop: 07.08.2019	
Contact: Artwork.Management@essity.com		Replaces Artwork ID: -	
Contact Address: BSN medical GmbH Quickbornstrasse24 20253 Hamburg Germany		Technical Drawing for print ID: -	
Ref Number(s): 34802-33		Comment on Diecut: Die cut for control purpose only – not for production purpose. Please use current die cut, as agreed with BSN medical. Technical proof, not binding in colour.	
Languages: DNK; FIN; FRA; DEU; ITA; NLD; NOR; PRT; ESP; SWE; GBR; TUR; POL; SVN; RUS; ISL; SAU; HUN; ROU; CZE; ISR			

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BSN-JOBST® GmbH